



CLEANING INSTRUCTIONS

NEBULIZER

MATERIAL REQUIRED: LIQUID DISH DETERGENT WATER
WHITE VINEGAR CLEAN CONTAINER

PROCEDURE:

AFTER EACH NEBULIZER TREATMENT

1. DISASSEMBLE ALL PARTS AS INSTRUCTED
2. RINSE PARTS WITH WARM RUNNING WATER
3. SHAKE EXCESS WATER AND ALLOW PARTS TO DRAIN DRY ON A CLEAN TOWEL
4. STORE IN A DUST FREE AREA

AT THE END OF THE DAY

1. DISASSEMBLE ALL PARTS AS INSTRUCTED
2. WASH IN WARM SOAPY WATER
3. RINSE WELL IN WATER
4. SHAKE EXCESS WATER AND ALLOW PARTS TO DRY ON A CLEAN TOWEL

DISINFECTING

(TO BE PERFORMED 2 - 3 TIMES PER WEEK)

1. AFTER FOLLOWING THE END OF THE DAY ROUTINE, SOAK THE CIRCUIT PIECES FOR 20-30 MINUTES IN A 1/3 VINEGAR SOLUTION. THIS SOLUTION IS MADE BY COMBINING 1 CUP VINEGAR AND 3 CUPS WATER
2. THOROUGHLY RINSE ALL PARTS WITH WATER
3. DRAIN DRY ON A CLEAN TOWEL AND STORE PROPERLY

NOTE: THESE CLEANING INSTRUCTIONS SUGGESTED BY RIVERSIDE MEDICAL, INC. THE MANUFACTURER'S CLEANING SUGGESTIONS CAN BE FOUND IN THE MANUAL THAT COME WITH EACH NEBULIZER.